

Premilene[®]

The success of a strong, smooth and flexible monofilament
Non-absorbable monofilament polypropylene suture



Sutures

Premilene®

The success of a strong, smooth and flexible monofilament

Premilene® thread

Premilene® is composed of an isotactic polypropylene, which presents an absolutely regular and longitudinal molecular structure. This composition together with our special extrusion process provides Premilene® with:

- Prolonged tensile strength retention.
- Smoothness thanks to its monofilament structure, causing marginal drag through the tissue.
- Adequate elasticity and elongation values, which provide a stable and secure knot.
- Pliability for easy knot tying and comfortable continuous suturing.

Premilene® needles

- Made of 300 series stainless steel to provide superior breaking resistance and ductility.
- Improved needle penetration thanks to its geometry and surface treatment.
- Special needle range for cardiovascular, plastic and ophthalmic surgery (DSMP, HRC, CV Pass needles).



Easyslide




CV Pass



CV Pass Easyblack




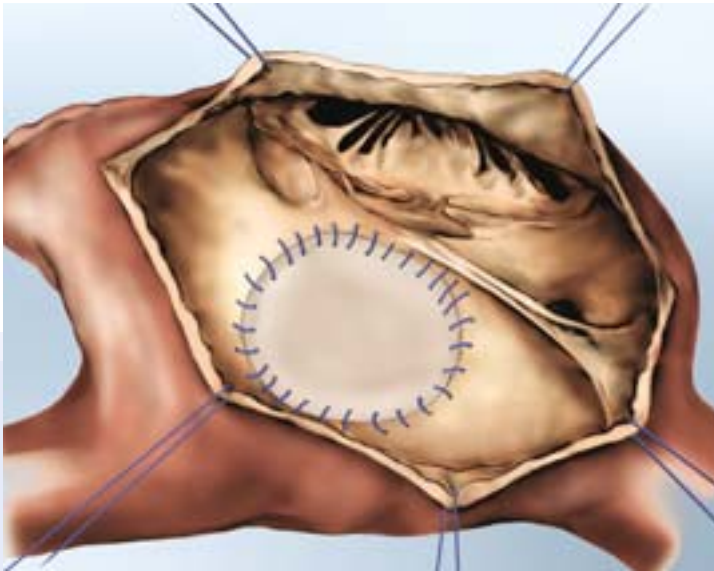

Indications

Premilene® is indicated for general soft tissue approximation and/or ligation, where the use of non-absorbable sutures is required.

- Cardiovascular surgery
 - Premilene® is specially indicated for anastomoses in cardiothoracic surgery
- Vascular surgery
 - Ideal for peripheral arterial and venous suturing
- Plastic surgery procedures
- Skin closure (specially intracutaneous)
- General surgery
- Ophthalmology
- Microsurgery (nerve and vessel reconstruction)

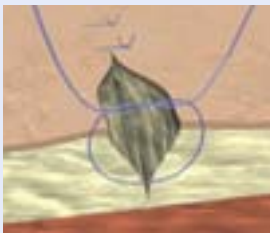


Skin closure



Plastic surgery



Ophthalmology

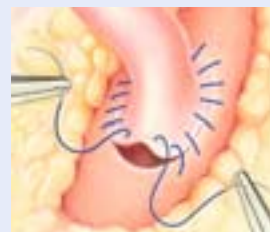

Cardiovascular surgery



Premilene®



The success of a strong, smooth and flexible monofilament

Available with CV Pass and CV Pass Easyblack

- A great alternative for vascular and cardiovascular microsurgery.
- The new generation of needles with high bending strength and tissue protection.



CV Pass and CV Pass Easyblack

- Fine, strong tip replaces trocar bevel

- Premium surface treatment improves penetration capacity even after several stitches
- Extreme stability under high strain

- Minimized transition from the needle to the thread – avoids vessel leaking
- Reinforced zone tapers off gradually
- **Semi square body design:**
 - Improves bending strength resistance
 - Enhances stability in the needleholder



A complete offer for demanding surgeons



A unique and innovative suture packaging system by B. Braun

B. Braun has a long history of suture expertise and the use of state of the art technology. Using this expertise and our commitment to constant improvement in sutures, packaging and needles.

RacePack® has been developed together with the OR staff to provide a functional packaging system which meets the requirements of the surgical team.



RacePack®

New RacePack® reduces up to 40 % the memory effect compared to similar packaging in the market.



Aesculap Instruments



Optilene® mesh



Silver Graft



Uni-Graft® K DV

Complementary B. Braun products

We strongly recommend using the B. Braun graft together with Premilene®, some of the most outstanding being Uni-Graft® K DV and Silver Graft.

The successful Optilene® Mesh as well as the Aesculap instruments are also great products to be used together with Premilene®.

Contact your local sales representative for further information on these products.



Premilene®






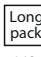
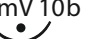



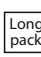
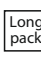

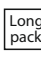

A complete offer for demanding surgeons

Product overview

Structure	Monofilament
Colour	Blue
Chemical composition	Polypropylene
Coating	Uncoated
Origin	Synthetic
Sizes	USP 10/0 (0.2 metric) to USP 2 (5 metric)
Type of absorption	Non-absorbable
Indication	Vascular and cardiovascular surgery, plastic and reconstructive surgery, skin closure, neurosurgery, Ophthalmology.
Sterilization	Ethylene oxide

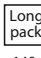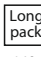

Needle-thread combination

Needle			Thread	USP metric						
length mm	angle	diameter microns	length cm colour	8/0 0.4	7/0 0.5	6/0 0.7	5/0 1	4/0 1.5	3/0 2	2/0 3
3/8 circle round bodied micro vascular needle with taper point										
CV Pass easyblock	2xDRmV 6b 		75 Blue	G2095086						
		135°	140μ							
			60 Blue	G0095040						
			2 x 45 Blue	G0095031						
		135°	140μ							
CV Pass easyblock	2xDRmV 8b 		75 Blue	G0095060						
		135°	140μ							
CV Pass easyblock	DRmV 10b 		Blue				G2095469			
		135°	200μ							
CV Pass easyblock	2xDRmV 10b 		75 Blue		G2090989	G2095139				
		135°	200μ							
			75 Blue	G0090064	G0095138					
			2 x 60 2 x 75 Blue		G0095072 G0090988	G0095108				
		135°	200μ							
CV Pass easyblock	DRmV 13b 		75 90 Blue				G0095077 G0095076			
		135°	320μ							
			75 90 Blue				G0095079 G0095078			
		135°	380μ							

C Box of 36 units


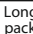






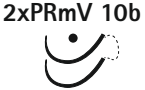



B Box of 24 units

G Box of 12 units




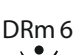
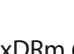


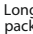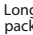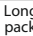













M Box of 6 units

Premilene®

Needle-thread combination

Needle			Thread	USP metric						
length mm	angle	diameter microns	length cm colour	8/0 0.4	7/0 0.5	6/0 0.7	5/0 1	4/0 1.5	3/0 2	2/0 3
3/8 circle round bodied micro vascular needle with taper point										
			60 Blue				G2095201			
	135°	320μ								
			75 90 Blue				G0095133 G0095132			
	135°	320μ								
			75 90 Blue				G0095135 G0095134			
	135°	380μ								
			2 x 60 2 x 75 2 x 90 Blue				G0095269 G0095234 G0095224			
	135°	320μ								
			2 x 90 Blue				G0095225			
	135°	380μ								
Progressive curved micro vascular needle with taper point										
			75 Blue		G2095153	G2090670				
	95°/90°	200μ								
1/2 round bodied micro-needle with taper point										
			30 Blue				G2090740			
	180°	200μ								
			45 Blue				G2090742	G2090743		
	180°	200μ								

Needle-thread combination

Needle			Thread length cm colour	USP metric						
length mm	angle	diameter microns		10/0 0.2	9/0 0.3	8/0 0.4	7/0 0.5	6/0 0.7	5/0 1	4/0 1.5
3/8 round bodied micro-needle with taper point										
	135°	70µ	10	G0090781						
			Blue							
	135°	100µ	15	G0090782						
			Blue							
	135°	100µ	45		G0090482					
			Blue							
	135°	140µ	15		G0090783					
			Blue							
	135°		45			G2090780				
			60			G2090609				
			75			G2095035				
	135°		45			G2090025				
			Blue							
	135°		15			G0090753				
			Blue							
			45			G2090983				
	135°		60			G2090758				
			75			G2090026				
			Blue							
	135°		75				G2090784	G2090785		
			Blue							
	135°		60			G2095106	G2090986	G2090987		
			75			G2095107	G2090786	G2090787		
			Blue							
	135°	200µ	2 x 75					G0095082		
			Blue							
3/8 round bodied micro vascular needle with short cutting point										
	135°		75				G2095685	G2095672		
			Blue							

C Box of 36 units




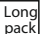
















B Box of 24 units

G Box of 12 units























M Box of 6 units

Premilene®

Needle-thread combination

Needle			Thread	USP metric						
length mm	angle	diameter microns	length cm colour	10/0 0.2	9/0 0.3	8/0 0.4	7/0 0.5	6/0 0.7	5/0 1	4/0 1.5
Progressive curved round bodied micro vascular needle with short cutting point										
	2xPRCmV 10b 	135°	 200µ	60				G2090647		
				75				G2090669	G2090668	
		135°	 200µ	60				G0095657		
3/8 round bodied micro-needle with short cutting point										
	2xDRCm 6 	135°	 140µ	60			G2090702			
				75			G2095277			
	2xDRCm 8 	135°	 140µ	60			G2095249			
				75			G2095284			
	DRCm 10b 	135°	 200µ	75				G2090768		
	2xDRCm 10b 	135°	 200µ	75			G2090694	G2090693		
Progressive curved round bodied micro-needle with short cutting point										
	2xPRCm 6 	100°/95°	 140µ	60			G2090698			
1/2 circle micro-lancet needle										
	2xHLM 6 	160°	150µ	30	G0090775					
3/8 circle micro-lancet needle										
	2xDLm 6 	140°	150µ	30	G0090591					
	DLm 6s 	137°	200µ	15		G0090594				
	2xDLm 6s 	137°	200µ	30	G0090596	G0090597				

Needle-thread combination

Needle			Thread	USP metric						
length mm	angle	diameter microns	length cm colour	10/0 0.2	9/0 0.3	8/0 0.4	7/0 0.5	6/0 0.7	5/0 1	4/0 1.5
1/8 circle micro-lancet needle										
	44°	150µ	150 Blue		G0090778					
Needle			Thread	USP metric						
length mm			length cm colour	8/0 0.4	7/0 0.5	6/0 0.7	5/0 1	4/0 1.5	3/0 2	2/0 3
1/2 round bodied needle										
	2xHR 6 		45 Blue		C2090803					
	2xHR 8 		60 Blue			C2090865				
			2 x 60 Blue			G0095865	G0095866			
	2xHR 10f 		2 x 75 Blue			G0095858				
1/2 circle round bodied needle										
	HR 10 		75 Blue			C2090402				
	2xHR 10 		75 Blue		C2090867	C2090868	C2090869			
	2xHR 13f 		45 75 Blue			C2095120 C2095123	C2095124			
			2 x 75 Blue			G0095127	G0095128			
	HR 13 		2 x 75 Blue				G0095095			

C Box of 36 units


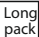






B Box of 24 units

G Box of 12 units


















M Box of 6 units

Premilene®

Needle-thread combination

Needle		Thread	USP metric								
			8/0 0.4	7/0 0.5	6/0 0.7	5/0 1	4/0 1.5	3/0 2	2/0 3		
length mm	length cm colour										
1/2 circle round bodied needle											
CV Pass	2xHR 13		60			C2095935					
			75			C2090953	C2090954				
			90					C2090750			
			Blue								
			2 x 60				G0090146				
			2 x 75			G0090625					
			Blue								
CV Pass	2xHR 13b		75			C2095448	C2095449				
			Blue								
CV Pass	2xHR 17f		75				C2095501				
			Blue								
CV Pass	HR 17		75				C2090012	C2090013	C2090014		
			Blue								
CV Pass	2xHR 17		75					C2090503	C2090502		
			90				C2090901	C2090902	C2090903		
			Blue								
			90					C0090837			
			Pledgets 6x3 mm Firm	Blue							
			2 x 75						G0095183		
			2 x 90				G0090642	G0090194	G0090644		
			Blue								
CV Pass	2xHR 17b		75				C2095460	C2095461			
			90				C2095492	C2095493	C2095494		
			Blue								

Needle-thread combination

Needle length mm	Thread length cm colour	USP metric						
		8/0 0.4	7/0 0.5	6/0 0.7	5/0 1	4/0 1.5	3/0 2	2/0 3
1/2 circle round bodied needle								
 2xHR 20 	 90 Blue				C2090223	C2090338	C2090426	
 2xHR 20b 	 90 Blue					C2095526		
 HR 22 	 75 Blue					C2090029	C2090030	
 2xHR 22 	 75 Blue				C2090975	C2090863 C2090976	C2090860 C2090977	C2090978
	 40 Blue						C0095387	
	 2 x 90 Blue					G0095620		
 2xHR 22b 	 75 90 Blue					C2095507 C2095512	C2095508 C2095513	

C Box of 36 units










B Box of 24 units

G Box of 12 units




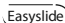

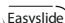










M Box of 6 units

Premilene®

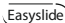Needle-thread combination

Needle		Thread	USP metric							
			7/0 0.5	6/0 0.7	5/0 1	4/0 1.5	3/0 2	2/0 3	0 3.5	
length mm		length cm colour								
1/2 circle round bodied needle										
CV Pass	HR 26	 75 Blue					C2090040	C2090041	C2090042	C2090043
		 Pledgets 3x3 mm Firm	20 Blue					C0090495		
		 Pledgets 3x3 mm Soft	20 Blue					C0095417		
CV Pass	2xHR 26	 75 Blue			C2090906	C2090707	C2090708	C2090087		
		 Pledgets 6x3 mm Firm	90 Blue				C2090907	C2090908	C2090909	
			120 Blue				C2090709	C2095710		
		 2 x 90 Blue				G0090633	G0090631			
CV Pass EasyLock	2xHR 26b	 75 Blue				C2095557		C2095723		
		 90 Blue					C2095558			
		 2 x 90 Blue					G0095993			

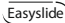Needle-thread combination

Needle length mm	Thread length cm colour	USP metric						
		4/0 1.5	3/0 2	2/0 3	0 3.5	1 4	2 5	3 6
1/2 circle round bodied needle								
  	75 90 Blue		C2090015	C2095442	C2095443	C2095452 C2095458		
 	75 90 Blue			C0090032	C0095444		C0090449	
  "Take Off" 	4 x 45 Blue				G0090073			
  	75 Blue		C2090046	C2090047	C2090048	C2090049		
 	90 Blue		C0090001	C0090002				
  	75 Blue				C2090092	C2090093		
	75 Blue				C0090092	C0090093		

C Box of 36 units



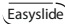











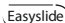


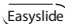


B Box of 24 units

G Box of 12 units



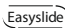
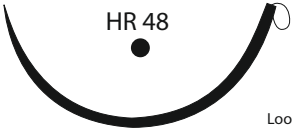
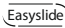
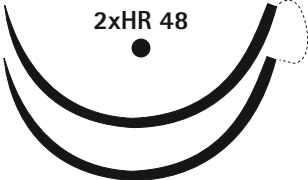
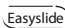

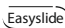


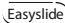


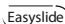M Box of 6 units

Premilene®

Needle-thread combination

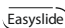Needle length mm	Thread length cm colour	USP metric						
		4/0 1.5	3/0 2	2/0 3	0 3.5	1 4	2 5	3 6
1/2 circle round bodied needle								
 HR 37 	75 Blue			C0090051	C0090052	C0090053		
 2xHR 37 	90 120 Blue		C0090942 C0095401	C0090943 C0090973				
 2 x 90 Blue		G0095727	G0095728					
 HR 37s 	75 120 Blue			C2090151 C2090154	C2090152	C2090153		
 HR 37s 	4 x 45 Blue				G0090057			
 2xHR 37s 	120 Blue		C0090971	C0090972				
 120 Blue				G0090972				
 HR 40 	100 150 Blue				C2090099	C2090100 C2095130		
 HR 40s 	75 100 Blue				C2090199	C2090200 C2090300		

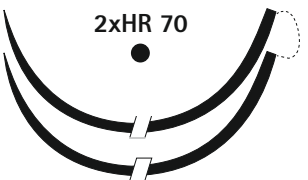Needle-thread combination

Needle length mm	Thread length cm colour	USP metric						
		4/0 1.5	3/0 2	2/0 3	0 3.5	1 4	2 5	3 6
1/2 circle round bodied needle								
 HR 48 	75 120 150 Blue				C2095445	C2090170 C2095146 C2090333		
 HR 48  Loop	150 Blue				C0090332	B0095081		
 2xHR 48  Long pack	120 Blue		G0090931	G0090932				
 HR 65 	120 150 Blue					B0090485	B0095585	
 HR 65  Loop	100 240 Blue					B0095578 B0095674		
 2xHR 70 	75 Blue					B0090841		
 HR 76 	150 Blue					B0095587		

C Box of 36 units















B Box of 24 units

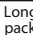G Box of 12 units

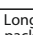

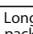
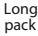




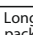


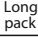
M Box of 6 units

Premilene®

Needle-thread combination

Needle		Thread	USP metric							
			8/0 0.4	7/0 0.5	6/0 0.7	5/0 1	4/0 1.5	3/0 2	2/0 3	
length mm		length cm colour								
3/8 circle round bodied needle										
CV Pass	2xDR 6 	 Blue	45	C2090880	C2095014					
			60	C2090712	C2095015					
			75		C2095286					
		 Blue	2 x 45	G0095841						
			2 x 60	G0090357						
CV Pass	2xDR 8f 	 Blue	45	C2090107	C2090079					
			60	C2090105	C2090088					
			2 x 45	G0090118	G0090124					
			2 x 60	G0090121						
		 Blue	2 x 75		G0090130					
CV Pass	2xDR 8 	 Blue	45	C2090703	C2090685	C2090686				
			60	C2090887	C2090888	C2090889				
			75	C2090004	C2090885	C2090886				
			2 x 45	G0090361	G0090362	G0090399				
			2 x 60	G0090363	G0090364					
		 Blue	2 x 75	G0090892	G0095080					
CV Pass	DR 10f 	 Blue	75		C2095084	C2095431				
CV Pass	2xDR 10f 	 Blue	45	C2090126	C2090134	C2095565				
			60	C2090128	C2090135	C2095477				
			75	C2090131	C2090136	C2095430				
			90			C2095576				
			2 x 45		G0090218	G0095566				
			2 x 60		G0095485	G0095801				
		 Blue	2 x 75	G0090216	G0095429	G0095847				

Needle-thread combination

Needle		Thread	USP metric								
			8/0 0.4	7/0 0.5	6/0 0.7	5/0 1	4/0 1.5	3/0 2	2/0 3		
length mm	length cm colour										
3/8 circle round bodied needle											
CV Pass	2xDR 10 	 Blue	45		C2090614	C2090615					
			60		C2090714	C2090715	C2090522				
			75		C2090914	C2090915	C2090913				
			90		C2090799	C2090815					
			Blue								
		 Pledgets 3x3 mm Soft	75			C0095306					
			Blue								
			2 x 45				G0090636				
			2 x 60		G0090611	G0090613					
			2 x 75		G0095810	G0095811					
 Long pack	4 x 60		G0090371								
	4 x 75		G0090373								
	Blue										
CV Pass	2xDR 10s 	 Blue	45		C2095600						
CV Pass	DR 12 	 Blue	45			C2090114					
			75		C2090529	C2090530	C2090531				
CV Pass	2xDR 12 	 Blue	45		C2090818	C2090816					
			60		C2090828	C2090705	C2090706				
			75		C2090922	C2090916	C2090917				
			90			C2090923	C2090924				
			Blue								
		 Pledgets 6x3 mm Firm	75				C0090825				
			Blue								
			2 x 45		G0095515	G0090850					
			2 x 60			G0090651					
			2 x 75			G0095818	G0090380				
 Long pack	2 x 90				G0090654						
	Blue										

C Box of 36 units




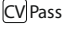









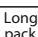
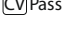






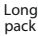

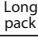



B Box of 24 units

G Box of 12 units





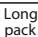

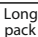






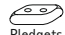
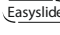





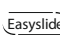


M Box of 6 units

Premilene®

Needle-thread combination

Needle		Thread	USP metric						
			8/0 0.4	7/0 0.5	6/0 0.7	5/0 1	4/0 1.5	3/0 2	2/0 3
3/8 circle round bodied needle									
 DR 13 		75						C2095932	
		Blue							
 2xDR 13 		45		C2095885	C2095886	C2095887			
		60		C2095894	C2095895				
		75		C2095905	C2095906	C2095907			
		90			C2095938	C2095939			
		Blue							
			75			G0095472			
 2xDR 13s 		2 x 45			G0095957				
		2 x 60			G0095970				
		2 x 75			G0095979				
		Blue							
 2xDR 13s 		60		C2095029	C2095032	C2095033			
		75		C2095024	C2095026	C2095027			
		90				C2095021			
		Blue							
			2 x 60		G0095012		G0095016		
		2 x 75			G0095011				
2 x 90					G0095009				
Blue									
 DR 18 		75				C2090119	C2090120	C2095057	
		Blue							
 2xDR 18 		75				C2090719	C2090720		
		90				C2090919	C2090920	C2095921	
		Blue							
			90					C0095215	
 2xDR 18 		Pledgets 6x3 mm Firm						C0095215	
		Blue							
 2xDR 18 		2 x 90				G0095659	G0090659		
		Blue							

Needle-thread combination

Needle		Thread length cm colour	USP metric						
length mm	8/0 0.4		7/0 0.5	6/0 0.7	5/0 1	4/0 1.5	3/0 2	2/0 3	
3/8 circle round bodied needle									
	2xDR 18b 	 Blue	75				C2095237	C2095238	
			90				C2095242	C2095243	
	DR 22 	 Pledgets 3x3 mm Firm	20						C0090492
			 Pledgets 3x3 mm Soft	20					
	2xDR 22 	 Blue	75						C2090723
			90				C2090822		
	DR 26 	 Blue	75						C2090132
			 Pledgets 3x3 mm Firm Knot	20					
	DR 32 	 Blue	75						C2090083
Needle		Thread length cm colour	USP metric						
length mm	6/0 0.7		5/0 1	4/0 1.5	3/0 2	2/0 3	0 3.5	1 4	
5/8 round bodied needle									
	FR 26 	 Blue	45					C2090156	
			75				C2095360	C2090066	
Round bodied ski needle									
	SKR 26 	 Blue	30			C2095421	C2090422		

C Box of 36 units




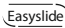

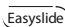

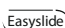











B Box of 24 units

G Box of 12 units


















M Box of 6 units

Premilene®

Needle-thread combination

Needle length mm	Thread length cm colour	USP metric						
		6/0 0.7	5/0 1	4/0 1.5	3/0 2	2/0 3	0 3.5	1 4
Progressive curved round bodied needle								
 PR 24 	 30 Blue				C2090243			
Straight round bodied needle								
 2xGR 65 	75 Blue					C0095350	C0095351	
 2xGR 70 	75 Blue					C0090771		
 2xGR 75 	75 Blue					C0090797		
1/2 round bodied needle with short cutting point								
 2xHRC 10 	 2 x 60 Blue	G0095794	G0095795					
 2xHRC 13 	 75 Blue	C2095039	C2090303					
	 2 x 75 Blue	G0095276	G0095043	G0095047				
 HRC 17 	 75 Blue			C2090561				

Needle-thread combination

Needle		Thread	USP metric							
			6/0 0.7	5/0 1	4/0 1.5	3/0 2	2/0 3	0 3.5	1 4	
length mm		length cm colour								
1/2 round bodied needle with short cutting point										
CV Pass	2xHRC 17 	 90 Blue		C2090570	C2090571	C2090800	C2095052			
		90 Blue		C0090570	C0090571		C0095052			
		 90 Blue			C0095367					
		 2 x 90 Blue		G0095168	G0095169	G0095182				
CV Pass	2xHRC 17b 	 90 Blue		C2095580	C2095581					
CV Pass	2xHRC 18 	 90 Blue				C2095372	C2095373			
CV Pass	HRC 22 	 75 Blue			C2090565	C2090566	C2095572			
CV Pass	2xHRC 22 	 90 Blue			C2090580	C2090581				
		 2 x 90 Blue			G0095162	G0095163				
CV Pass	2xHRC 22b 	 90 Blue			C2095592	C2095593				
CV Pass	HRC 26 	 75 Blue				C2090473		C2095796		

C Box of 36 units


















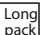


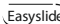




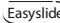

B Box of 24 units

G Box of 12 units







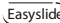













M Box of 6 units

Premilene®

Needle-thread combination

Needle	Thread	USP metric								
		length mm	length cm	6/0	5/0	4/0	3/0	2/0	0	1
	colour	0.7	1	1.5	2	3	3.5	4		
1/2 round bodied needle with short cutting point										
CV Pass  	 90				C2090811	C2090812	C2095825			
	120 Blue					C2090820	C2095140			
CV Pass  	 90					C0095377				
	Blue									
CV Pass  	 2 x 75				G0095759					
	2 x 90 Blue					G0095760	G0095761			
CV Pass  	 90				C2095607	C2095608				
	Blue									
CV Pass  	 90					C2095227	C2095228			
	Blue									
CV Pass  	 90						C2095670			
	Blue									
CV Pass  	90						C0090585			
	Blue									
Easyslide  	 75							C2095048	C2095734	
	Blue									
Easyslide  	120					G0095403				
	Blue									
Easyslide  	90						C0090831			
	Blue									

Needle-thread combination

Needle length mm	Thread length cm colour	USP metric						
		8/0 0.4	7/0 0.5	6/0 0.7	5/0 1	4/0 1.5	3/0 2	2/0 3
3/8 round bodied needle with short cutting point								
CV/Pass 	2xDRC 6 	45	C2095209	C2095219				
		60	C2095246	C2095251				
		75	C2095272	C2095288				
CV/Pass 	2xDRC 8 	60	C2095025	C2095046	C2090172			
		75	C2090169	C2090176				
		Blue						
CV/Pass 	2xDRC 10 	2 x 60	G0095178	G0095179				
		2 x 75	G0090168	G0095280				
		Blue						
CV/Pass 	2xDRC 10s 	60		C2090544	C2090543			
		75		C2090545	C2090546			
		Blue						
CV/Pass 	DRC 12 	2 x 60		G0090179	G0090202			
		2 x 75		G0095560	G0095561			
		Blue						
CV/Pass 	2xDRC 10s 	75		C2095324	C2095325			
		Blue						
CV/Pass 	DRC 12 	75			C2090550	C2090551		
		Blue						
CV/Pass 	2xDRC 12 	45		C2095241				
		60		C2090537	C2090524			
		75		C2090554	C2090555	C2090556	C2090557	
CV/Pass 	2xDRC 12 	Blue						
		75			G0095170			
		Blue						
CV/Pass 	2xDRC 12 	2 x 75			G0090185	G0095186		
		4 x 75			G0090183	G0090178		
		Blue						

C Box of 36 units














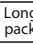



B Box of 24 units

G Box of 12 units








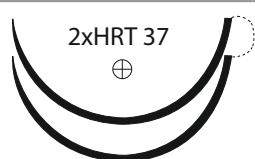



M Box of 6 units

Premilene®

Needle-thread combination

Needle		Thread length cm colour	USP metric						
length mm	8/0 0.4		7/0 0.5	6/0 0.7	5/0 1	4/0 1.5	3/0 2	2/0 3	
3/8 round bodied needle with short cutting point									
CV Pass	2xDRC 13 	 Blue	45			C2095173			
			60		C2095268	C2095271			
			75		C2095301	C2095175	C2095141		
			90		C2095303	C2095305	C2095171		
			Blue						
		 Blue	2 x 45			G0095174			
			2 x 75			G0095176	G0095177		
			Blue						
CV Pass	2xDRC 13s 	 Blue	75				C2095334		
			Blue						
Needle		Thread length cm colour	USP metric						
length mm	6/0 0.7		5/0 1	4/0 1.5	3/0 2	2/0 3	0 3.5	1 4	
3/8 round bodied needle with short cutting point									
CV Pass	2xDRC 15 	 Blue	75		C2095756				
			90		C2090728				
			Blue						
CV Pass	DRC 18 	 Blue	75		C2095164	C2095165	C2095166		
			Blue						
CV Pass	2xDRC 18 	 Blue	75		C2095194	C2095195			
			Blue						
CV Pass	2xDRC 18b 	 Blue	75		C2095247	C2095248			
			Blue						
CV Pass	2xDRC 22 	 Blue	90			C2090535	C2090536		
			Blue						
CV Pass	2xDRC 26 	 Blue	90			C2095630	C2095633		
			Blue						

Needle-thread combination

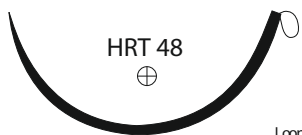Needle length mm	Thread length cm colour	USP metric						
		8/0 0.4	7/0 0.5	6/0 0.7	5/0 1	4/0 1.5	3/0 2	2/0 3
Progressive curved round bodied needle with short cutting point								
CV/Pass 	 60 75 Blue		C2095062 C2095064					
		 2 x 75 Blue	G0095053					
CV/Pass 	 120 Blue							C2095150
Needle length mm	Thread length cm colour	USP metric						
		5/0 1	4/0 1.5	3/0 2	2/0 3	0 3.5	1 4	2 5
1/2 round bodied needle with trocar point								
Easyslide 	 75 100 Blue					C2090154		C2090463
Easyslide 	120 Blue			C0090734	C0090735			
Easyslide 	 75 Blue					C2095463		
Easyslide 	150 Blue							B0090335

C Box of 36 units



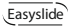

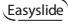













B Box of 24 units

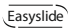G Box of 12 units

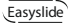


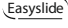

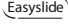

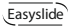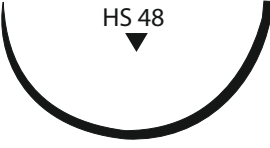




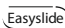


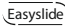




M Box of 6 units

Premilene®

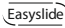Needle-thread combination

Needle length mm	Thread length cm colour	USP metric						
		6/0 0.7	5/0 1	4/0 1.5	3/0 2	2/0 3	0 3.5	1 4
1/2 round bodied needle with blunt point								
 HRN 45 	100 Blue							C0090643
1/2 reverse cutting needle								
 HS 15 	75 Blue			C0090259				
 HS 18 	 45 Blue		C2090264					
 HS 21 	 75 Blue				C2090461			
 HS 26 	 75 Blue				C2090269	C2090270	C2090271	
Needle length mm	Thread length cm colour	USP metric						
		4/0 1.5	3/0 2	2/0 3	0 3.5	1 4	2 5	3 6
1/2 reverse cutting needle								
 HS 40 	75 100  Blue				C2090281 C2095912	C2090287		
 HS 48 	100 Blue					C0090304		

Needle-thread combination

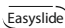Needle length mm	Thread length cm colour	USP metric						
		6/0 0.7	5/0 1	4/0 1.5	3/0 2	2/0 3	0 3.5	1 4
3/8 reverse cutting needle								
 DS 9 	 45 Blue	C2090204						
 DS 12 	 45 75 Blue	C2090206 C2095543	C2090207	C2090208	C2090205			
 DS 16 	 45 75 Blue	C2090211	C2090212 C2090512	C2090213 C2090513	C2090514			
 DS 19 	 30 45 75 Blue	C2095400	C2090219 C2090519	C2090420 C2090220 C2090520	C2090421 C2090221			
 DS 24 	 45 75 90 Blue		C2090233	C2090434 C2090234	C2090435 C2090235	C2090436 C2090236	C2090237 C2090072	
 DS 30 	 45 75 Blue				C2090446 C2090246	C2090447 C2090247	C2090248	C2090440 C2095200
 DS 35 	75 Blue				C2090230			

C Box of 36 units


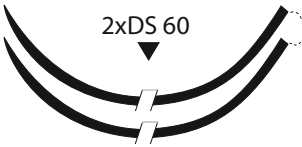



B Box of 24 units

G Box of 12 units




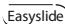





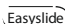





M Box of 6 units

Premilene®

Needle-thread combination

Needle length mm	Thread length cm colour	USP metric						
		4/0 1.5	3/0 2	2/0 3	0 3.5	1 4	2 5	3 6
3/8 reverse cutting needle								
 DS 39	75 Blue		C2090254	C2090255	C2090256	C2090257		
 2xDS 60	75 Blue			B0090751				
	75 Blue			C0090774				
 DS 76	3 x 50 Blue						B0090288	
 DS 90	75 Blue					C0090262		
Needle length mm	Thread length cm colour	USP metric						
		6/0 0.7	5/0 1	4/0 1.5	3/0 2	2/0 3	0 3.5	1 4
Straight reverse cutting needle								
 GS 60	75 Blue				C0090265	C0090266		

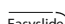Needle-thread combination

Needle length mm	Thread length cm colour	USP metric						
		8/0 0.4	7/0 0.5	6/0 0.7	5/0 1	4/0 1.5	3/0 2	2/0 3
3/8 reverse cutting needle with precision point								
 DSMP 7 	 45 Blue		C2090612					
 DSMP 11 	 45 Blue		C2090606	C2090607	C2090608			
Needle length mm	Thread length cm colour	USP metric						
		6/0 0.7	5/0 1	4/0 1.5	3/0 2	2/0 3	0 3.5	1 4
3/8 reverse cutting needle with precision point								
 DSMP 16 	 45 75 Blue	C2090628 C2090428	C2090629	C2090630	C2090481			
 DSMP 19 	 45 75 Blue		C2090675	C2090676	C2090677 C2090673	C2090678 C2090674		
 DSMP 24 	 45 75 Blue			C2090655 C2090680	C2090656 C2090681	C2090657 C2090682		

C Box of 36 units

B Box of 24 units

G Box of 12 units

M Box of 6 units



B. Braun Surgical SA | Carretera de Terrassa, 121 | 08191 Rubí | Spain
Phone +34 93 5 86 62 00 | Fax +34 93 6 99 63 30 | www.bbraun.es

Aesculap AG | Am Aesculap-Platz | 78532 Tuttlingen | Germany
Phone +49 7461 95-0 | Fax +49 7461 95-26 00 | www.aesculap.com

Aesculap – a B. Braun company

Subject to technical changes. All rights reserved.
This brochure may only be used for the exclusive
purpose of obtaining information about our
products. Reproduction in any form partial or
otherwise is not permitted.

Brochure No. B18302

0212/1/5